

# Instruction Manual



Please provide the instruction for use to the final user after installation.




Installation and maintenance must be performed by qualified technical personnel and following the assembly and maintenance instructions (see dedicated section “Assembly” and “Maintenance”) of this manual.

**Reproductions, duplications or translations, even in the form of extracts, are prohibited without prior permission from Multix Solutions S.r.l.**

# ASSEMBLY INSTRUCTION




The Trolley X1T is a Class I medical device according to the classification rules of Annex VIII and bears the “CE” marking in accordance with the Medical Device Regulation (MDR) (EU) 2017/745. It complies with IEC 60601-1:2005+AMD1:2012+AMD2:2020 and meets the general safety and performance requirements of Annex I of the aforementioned Medical Device Regulation.

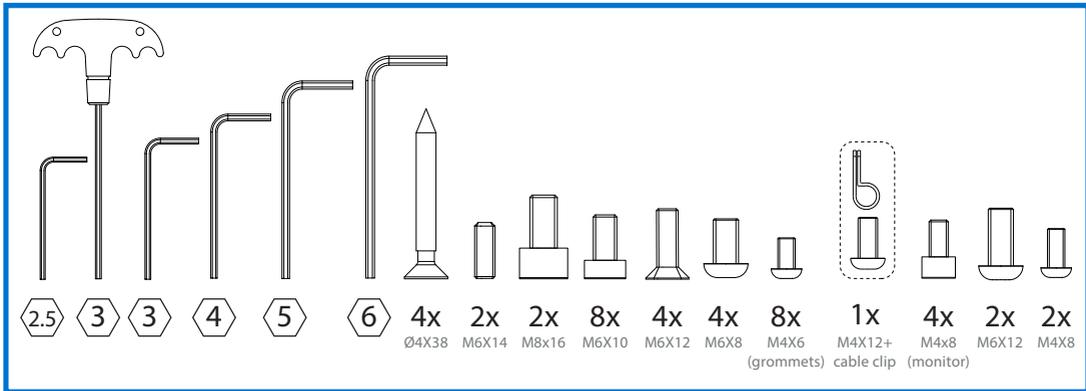

# ASSEMBLY INSTRUCTION

## 000CNSX1T - X1T TROLLEY

! If any accessory is present, please refer to the related instruction before assembling the trolley

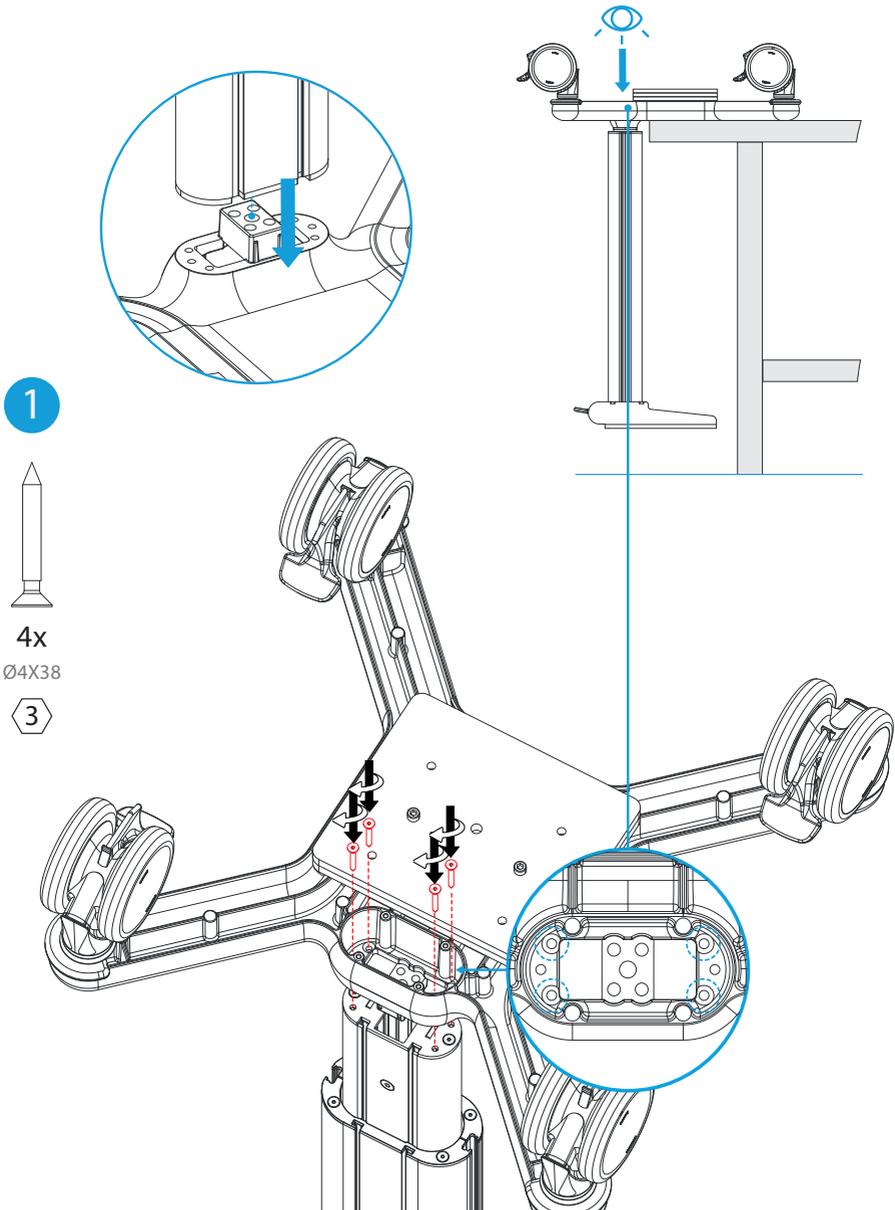

2




2x

M6X14




2x


M8x16

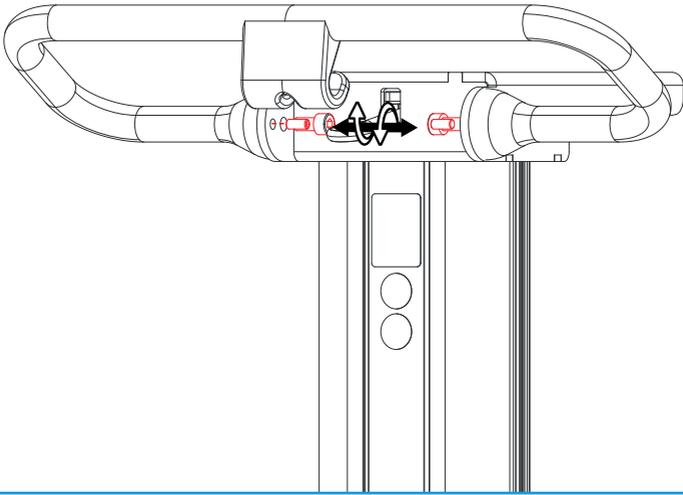

3

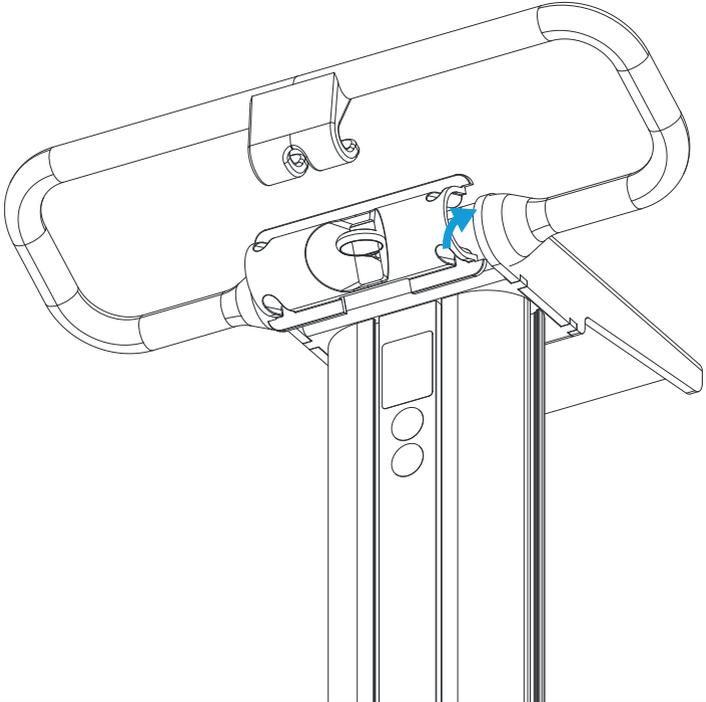


5

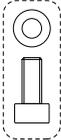

4x  
M6x18  
metal washer

5

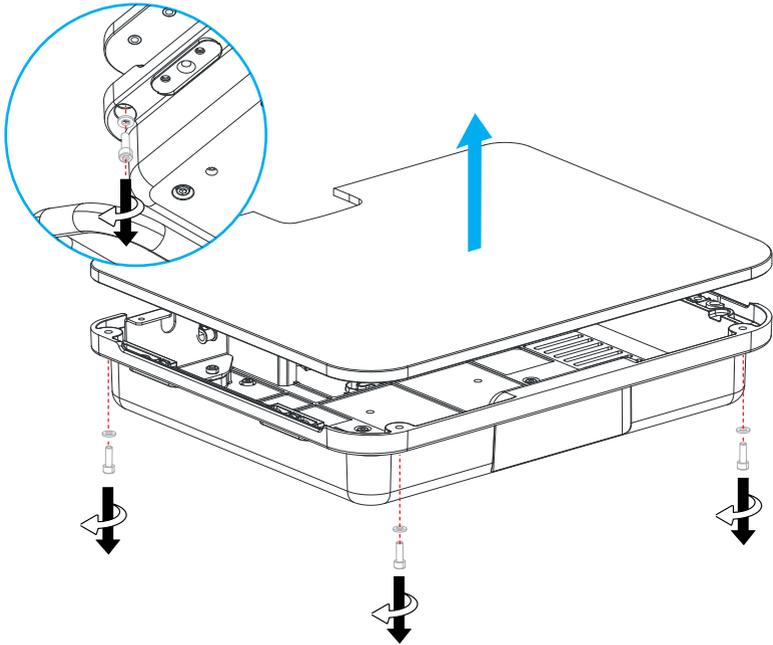


6



4x  
M6x12


4

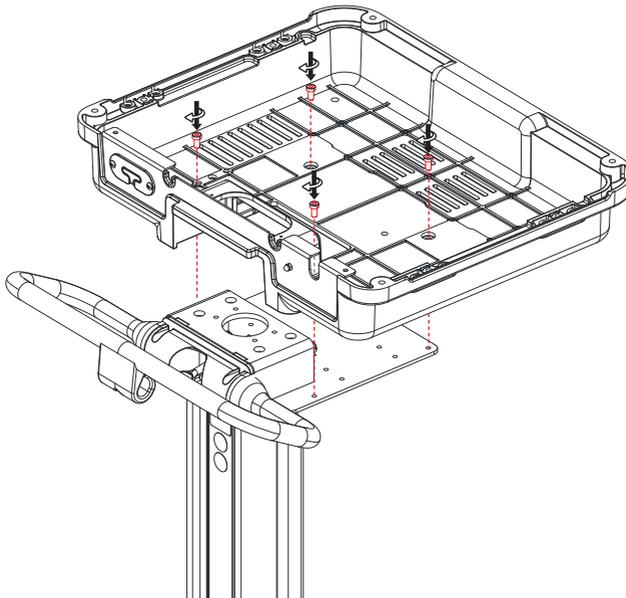

7




4x  
M6x12

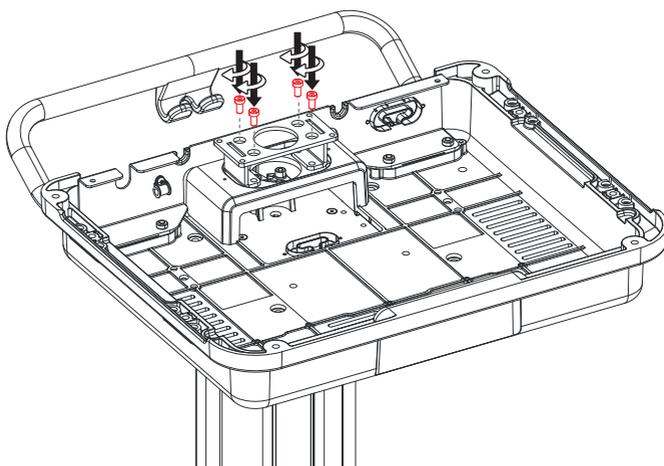


8



4x  
M6x10

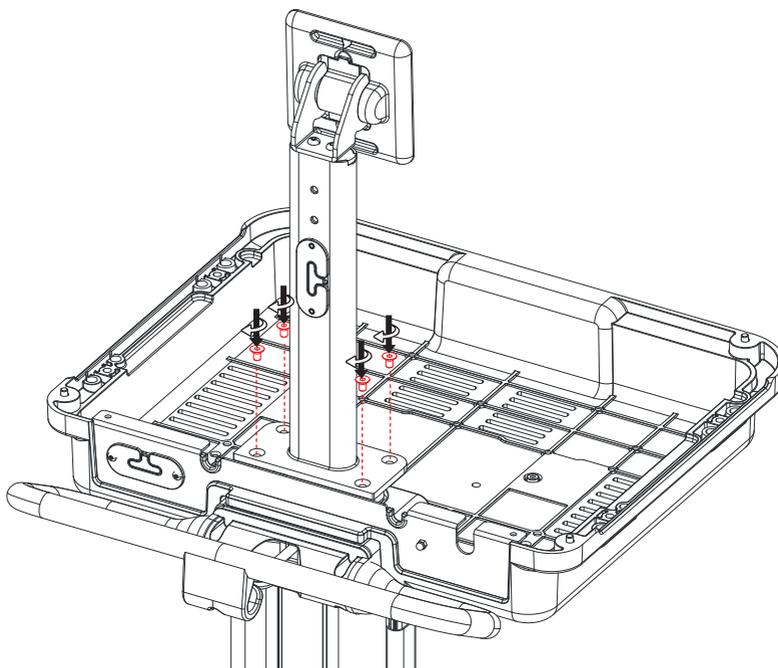


9

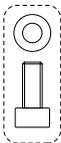

4x  
M6x18  
metal washer

5

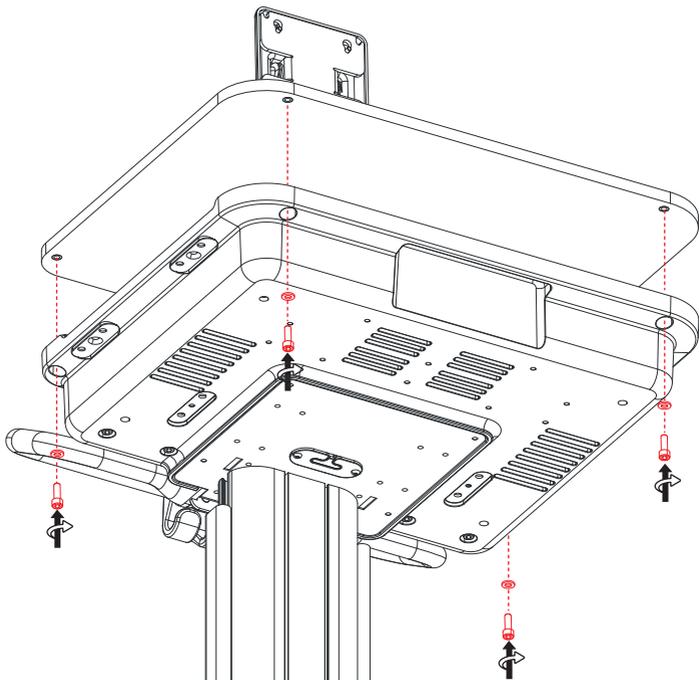



10



4x  
M6x8

4

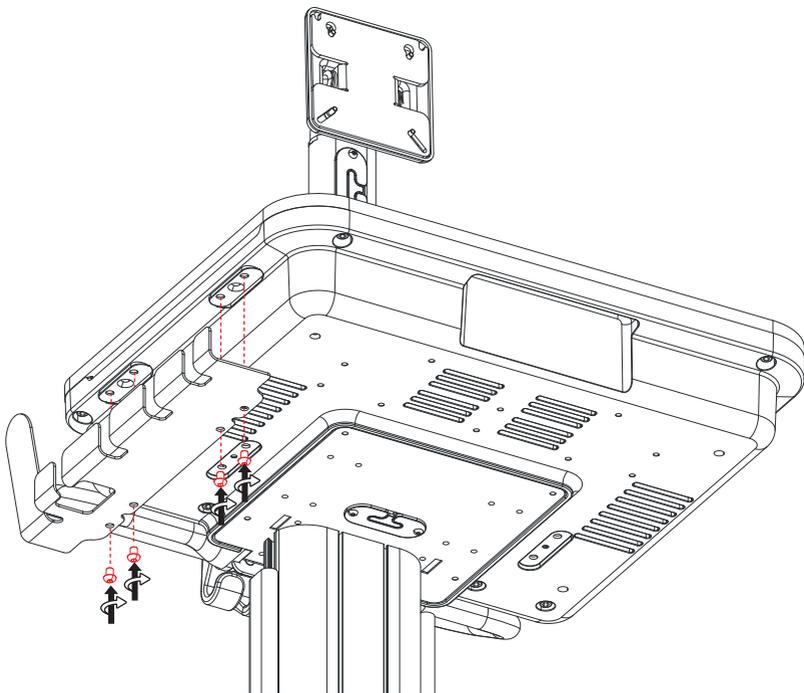

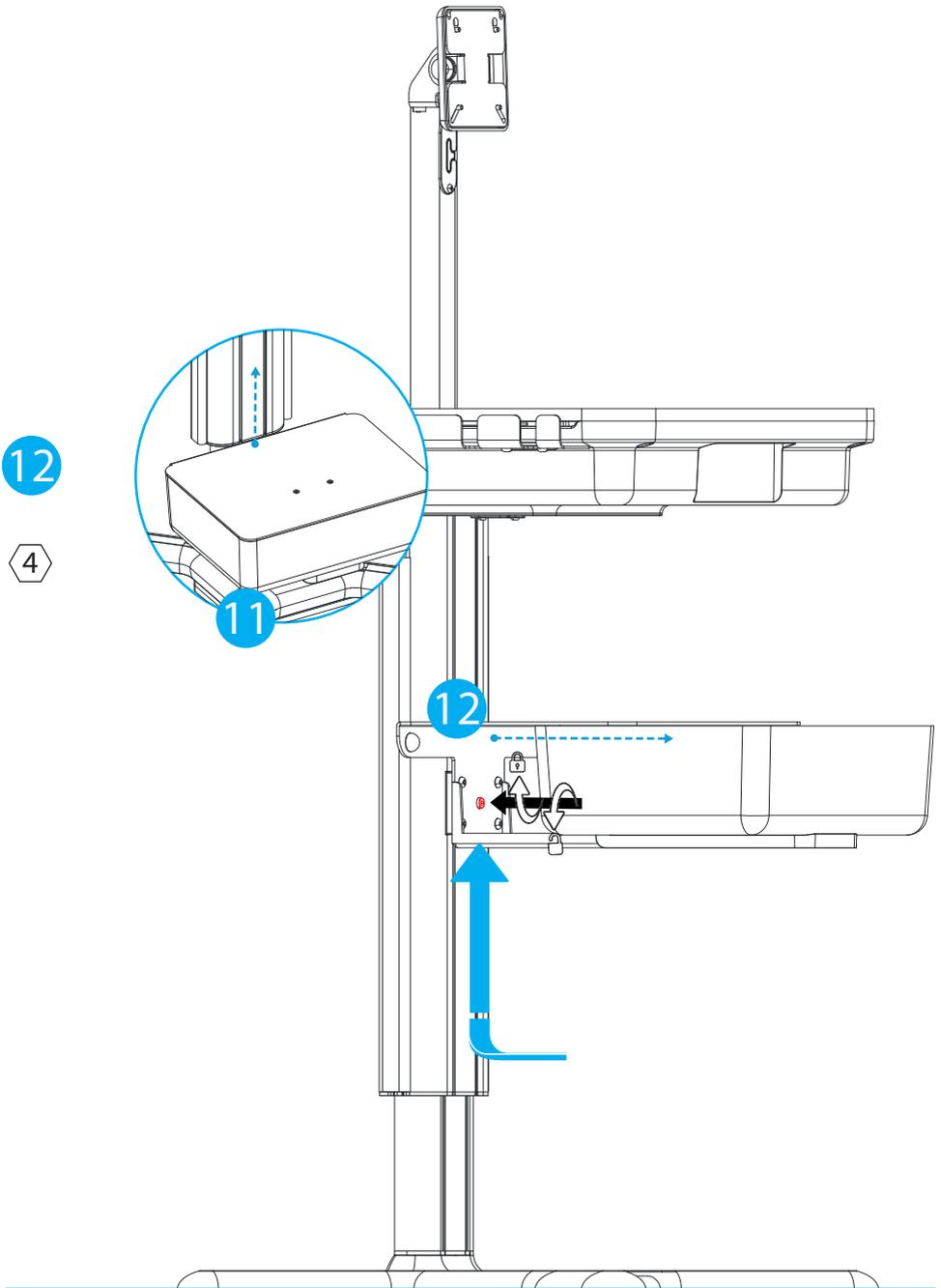


13



4x

M4X6

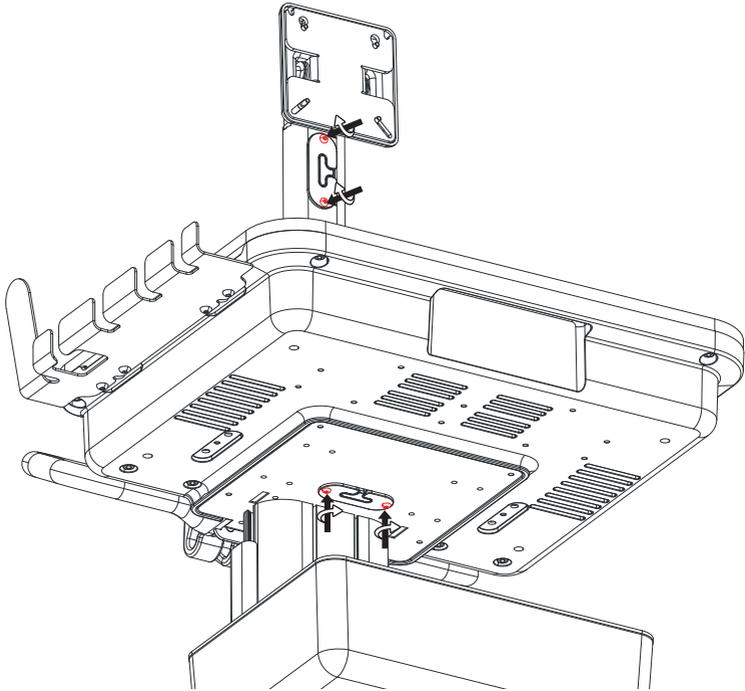


if needed, use the M4X6 to fix the grommets

14




1x

M4X12+  
cable clip

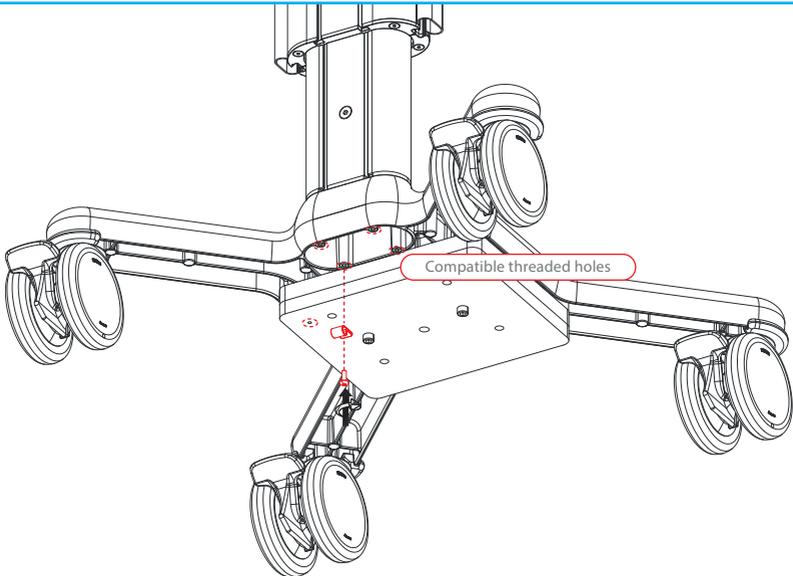

Compatible threaded holes

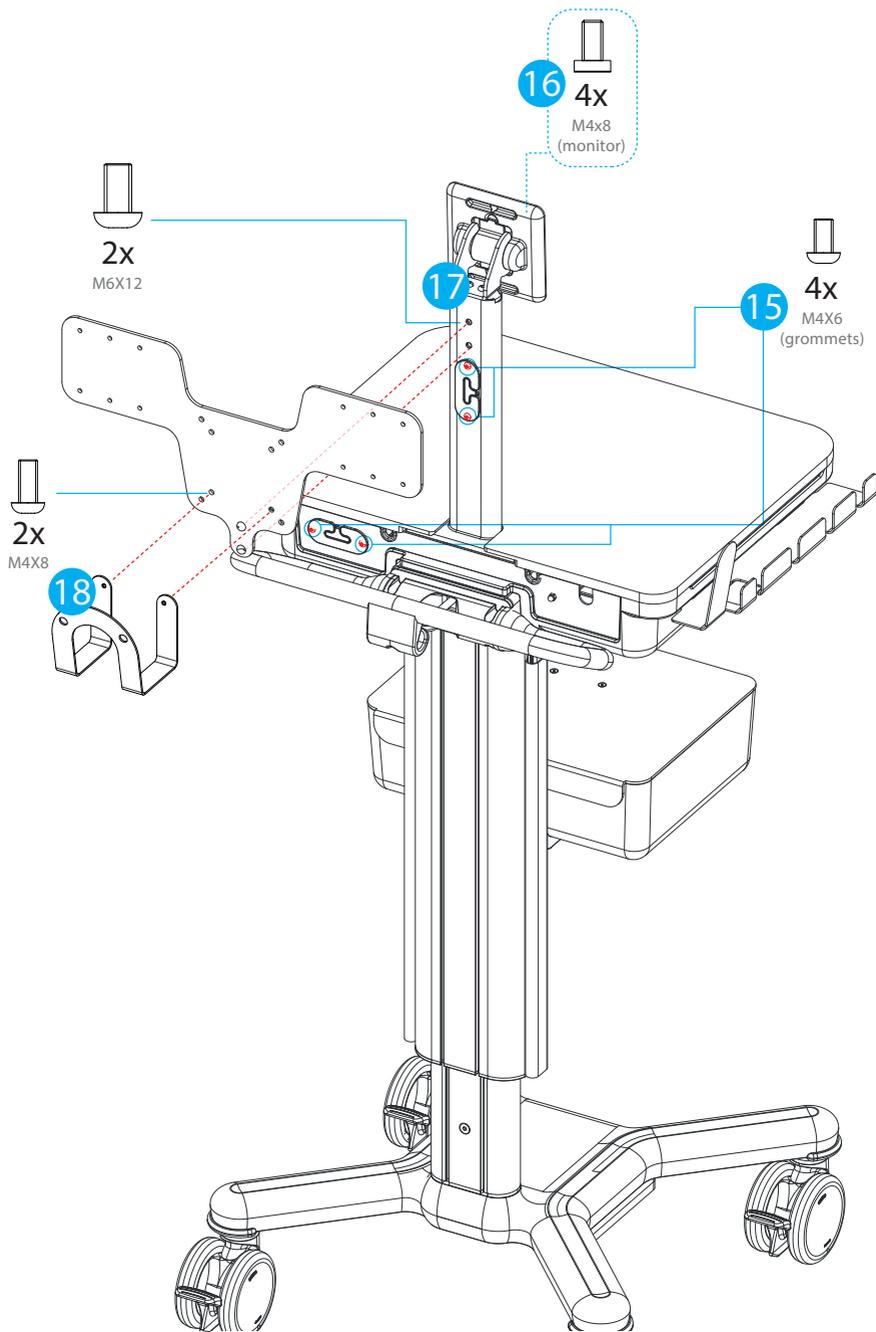

if needed, use the M4X6 to fix the grommets

# Important notices



Installation and maintenance must be performed by qualified technical personnel and following the assembly and maintenance instructions (see dedicated section “Assembly” and “Maintenance”) of this manual.

## INTENDED USE

The **X1T trolley - 000CNSX1T** enables the transport of the **an equipment and its accessories** between points of therapy in the hospital environment and limited care facility environment.

## GENERAL SAFETY OBLIGATIONS

- The Trolley instructions must be read before use.
- Keep the instructions for future reference.
- The product was created exclusively for the methods mentioned in the instruction manual. Any other use is to be considered improper.
- Tampering or damage caused by incorrect use of the cart leads to the immediate forfeiture of the guarantee.
- Multix Solutions is not liable for damage to persons and/or things due to improper use.
- When **an equipment** is mounted on the trolley, it will be responsibility **of the system installer** to ensure that the system complies with the current applicable standards, EN 60601-1 current version.



## GENERAL WARNING AND CAUTION

**The safety of the cart is guaranteed by carefully following the warnings below:**

- Make sure the cart matches the illustration in the manual.
- Check that it is not broken or damaged during transport.
- Check that the moving and locking parts such as wheels, brakes and (if present) the gas spring are functional.
- All four wheels of cart have to be locked, when the system and its accessories are assembled.
- When the cart is not being driven, ensure that the cart wheels are locked and that the system is not parked on uneven surfaces.
- All wheels of the cart have to be unlocked before the cart is pushed.
- To prevent the cart from tipping over, the handle must be held if the cart is driven over obstacles (for example power cords).
- Avoid an excessive force when pushing the cart.
- No changes to this device are permitted.




## LOAD INSTRUCTIONS

- IT IS FORBIDDEN TO SIT ON THE CART
- PUSHING PROHIBITED WHEN THE WHEELS ARE BLOCKED
- Follow the maximum loads as indicated in section “transport position and direction”
- Ensure that the total weight of the configuration mounted (added the accessories) does not exceed the overall weight stated on the label applied on the cart and on the next page.

# Important notices

## TRANSPORT POSITION AND DIRECTION

During transportation the trolley has to be lowered till the end of the gas spring , holding the handle run facing forward, as shown in the picture below:

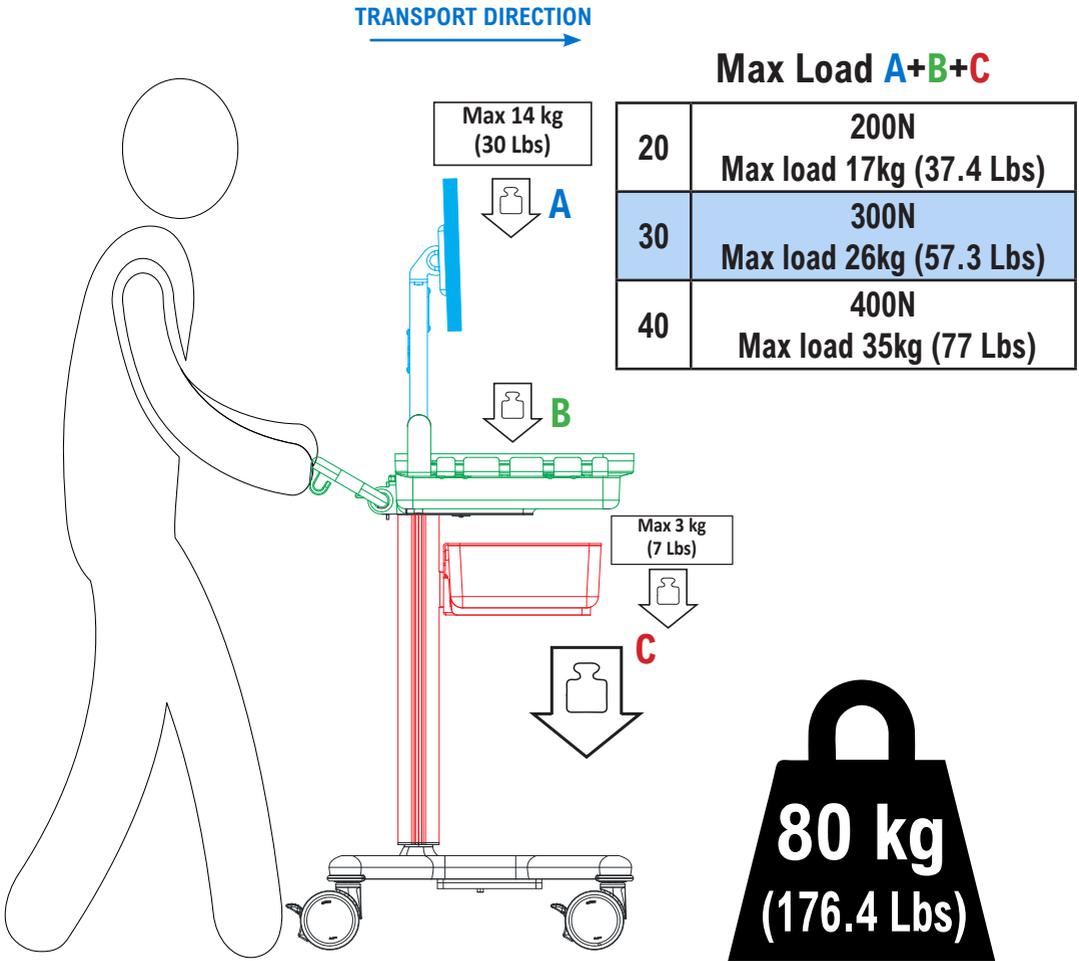

# Appendix



Installation and maintenance must be performed by qualified technical personnel and following the assembly and maintenance instructions (see dedicated section “Assembly” and “Maintenance”) of this manual.

## MAINTENANCE

Check all fixing points and mechanisms every 6 months.  
The product is only suitable for use in indoor environments.

## CLEANING

Regularly clean the metal and/or plastic surfaces with non-corrosive neutral pH detergents and a cloth/sponge/brush with soft bristles.  
Do not pour directly on surfaces liquid, chlorine-based or ammonia-based detergents, greasy and oily substances.

## SPARE PARTS

For spare parts refer to the device distributor (see Contact section)

## SERVICE LIFE AND DISPOSAL

The lifetime of the product is 10 years.  
The cart and the packaging shall be disposed according to local or national regulation.

<p>cartonbox and internal reinforcements</p>  <p>20 PAP</p> <p>CARDBOARD</p>	<p>internal packaging</p>  <p>04 PE-LD</p> <p>LOW-DENSITY POLYETHYLENE</p>
---	---

## GLOSSARY OF SYMBOLS

The following symbols are reported on the cart or in the instruction:

	Caution		CE-Marking		Catalogue number
	Operation instruction		Manufacturer		Model Number
	Medical Device		Batch code		Date of manufacture
	Humidity limitation		Orientation		Fragile, handle with care
	Keep dry		Keep away from sunlight		Temperature limit
	It's forbidden to sit on the cart		Pushing Prohibited		Distributed by
	Total weight				



**Distributed by: CNSystems Medizintechnik GmbH**



Reininghausstrasse 13  
8020 Graz  
Austria  
[www.cnsystems.com](http://www.cnsystems.com)

**Manufactured by: Multix Solutions s.r.l.**



Via Padre Rovagnati 2  
22066  
Mariano Comense (CO)  
Italy  
[www.multix-trolley.com](http://www.multix-trolley.com)  
[info@multix-trolley.com](mailto:info@multix-trolley.com)

